



SPOTLIGHT

on aged care



**INVESTIGATING
URINARY INCONTINENCE**



**TIPS ON CHOOSING
WOUND DRESSINGS**



**NEW TO
SSS AUSTRALIA**



**SPEECH
PATHOLOGY**



**BRAND NEW
BUDGET ELECTRODES**



**ZINC OXIDE &
SKIN INTEGRITY**

CONTACT YOUR PERSONAL ACCOUNT MANAGER



CAROL EDMOND

Logan, Gold Coast, Northern Rivers,
West Brisbane, & Northern Territory
Mobile: 0408 181 938

FIONA CAREY

Brisbane North, Wide Bay,
Capricorn, & Western Australia
Mobile: 0427 361 761



JEFF OSBOURNE

Metro & Regional NSW
Mobile: 0428 759 080

SHANNON JENNINGS

Brisbane South, Ipswich,
Toowoomba, & South Australia
Mobile: 0419 172 150

PAUL COOPER

Sunshine Coast, North QLD,
& Far North QLD
Mobile: 0482 629 882



AMAAN VADDADI

Metro & Regional VIC
Mobile: 0400 121 010



ITEM ATTRACTS EXTRA
SHIPPING CHARGES

Edition 3, 2024

Offers valid July 1, 2024 until September 30, 2024 unless stated otherwise

ABN: 77 168 062 308

www.sssaustralia.com.au



QUICK TIPS FOR CHOOSING A WOUND DRESSING

by Dr Michelle Gibb

Dressing selection can be overwhelming and it's no wonder with thousands of different dressings available. With such a wide variety of products to choose from, it's important that you know the types of dressings available, when and when not to use them, and how to use them effectively.

Fortunately, like medications, dressings typically fall into generic categories or classes. Understanding the generic category of the product, their actions, indications, contraindications, advantages and disadvantages for use, will help with decision making.

When evaluating the performance of your dressings, ask yourself these questions:

- Does the dressing stay intact and remain in place for an extended wear time?
- Prevent leakage between dressing changes?
- Does it provide a moist wound healing environment?
- Protect the periwound skin?
- Reduce wound-associated pain?
- Reduce or control wound odour?
- Does the person with the wound find the dressing conformable and comfortable?
- Is the dressing easy to apply and remove without causing trauma to the wound or periwound skin?
- Are the dressings cost-effective (remembering that cheap isn't always cost-effective)?
- Supported by evidence and resources?

If you're still not sure what dressings to select, then you might want to check out the **A to almost Z of Wound Dressings**. It contains information on the most common types of dressing products available with useful hints and tips to guide dressing selection. Order today!



Item #: 1504754 Each

\$30 ex GST

BACK TO BASICS WOUNDCARE WORKSHOP

presented by Dr Michelle Gibb

SYDNEY

Canada Bay Club
FRIDAY 19 JULY 2024

**SEATS STILL
AVAILABLE!**

Focusing on the basics of wound care we will be discussing treatment options for different types of common wounds, how to best address challenges in wound care, and working through ten case studies. The workshop also includes a 90-minute practical session.



\$10 TICKETS

Includes

- Lunch, morning tea, tea & coffee
- Digital certificate of attendance
- Goodie bag



SCAN HERE or visit
our website for more
information and to
purchase tickets

sssaustralia.com.au/events



SURFACE CLEANING



**SPECIAL
PRICING
FOR Q3
ONLY!**

V-Wipes Flat Pack Packet 80 Wipes

Clean and disinfect surfaces with V-Wipes' Instrument Grade Disinfectant 'low level. V-Wipes' unique pH-neutral formula and high-quality apertured fabric allow for fast, effective cleaning and disinfection of all surfaces within the healthcare environment. V-Wipes are intended for use on environmental surfaces and for non-critical medical devices such as hospital beds and theatre trolleys. V-Wipes are effective against the SARS-CoV-2 (COVID-19) Virus in just 60 seconds.

Item #: 1471554 Each

\$8⁴⁹ ex GST

Item #: 1470213 C12

\$99 ex GST



HiCare 2in1 Antibacterial Disinfect Wipes Hands/Face/Surface

HiCare™ 2 in 1 Antibacterial Disinfectant Wipes are your ultimate solution to eliminating 99.9% of harmful germs and bacteria. These versatile wipes are specifically crafted to efficiently eradicate germs, grease, dirt, and grime from both your body and various surfaces. Whether you're at home, school, work, or any place where people gather, these wipes make disinfection a breeze, leaving you feeling refreshed and moisturised.

Item #: 1507850 P100

\$6³⁰ ex GST



HiCare Antibacterial Disinfect Surface Wipes (Mild Lemon)

HiCare™ Antibacterial Disinfectant Surface Wipes are your ultimate defense against harmful germs and bacteria. These wipes are expertly designed to eliminate 99.9% of pesky germs, efficiently remove grease, and clean soiled hard surfaces. They are perfect for everyday use, whether you're at home, school, work, or any place where people gather, making disinfection a breeze right at the source.

Item #: 1507869 P100

\$6³⁰ ex GST



HealthSource Neutral Detergent Surface Wipes

Ready-to-use Neutral Detergent Surface Wipes for cleaning medical devices and surfaces in a medical/healthcare facility. Safe to use on multiple surface types pH Neutral, low foaming. Ideal for healthcare applications.

Item #: 1471724 P48

\$52⁵⁰ ex GST

Fragrance-free Non-tainting. Developed for the healthcare environment. Can be conveniently placed for timely use by staff 48 wipes per pack



Speedy Clean Wipes Flat Pack

Speedy Clean Wipes are excellent ready-to-use hospital grade, anionic neutral detergent wipes which leaves a streak free finish on surfaces. Ideal for cleaning surfaces and equipment in health-related environments. The single-use application improves infection control by avoiding cross contamination.

Item #: 1489895 P80

\$10¹⁰ ex GST



Clinell Sporicidal Wipes

Peracetic acid generating wipes for safe surface disinfection and the cleaning of non-invasive medical devices. Effective against spores in one minute.

Item #: 1367250 P25

\$24⁷⁵ ex GST

The safest, most trusted sporicidal wipes for high level cleaning and disinfecting of surfaces and non-invasive medical devices. Incredibly powerful, with 6 log spore kill count in just 1 minute, yet kind to surfaces thanks to patented, near-neutral pH formula. Proven to reduce C. difficile-associated disease by 72%.



Halyard 2n1 Hospital Grade Disinfectant Wipes

Environmental cleaning and disinfection are crucial in preventing transmission of infection in the healthcare environment.

Item #: 1524623 P100

\$8²⁵ ex GST

Quaternary Ammonium Compounds (QACs) are positively charged ions that bind to the negatively charged outer membranes of germs, breaking down the membrane resulting in the killing or inactivation of those germs.

Item #: 1524631 P200

\$16⁴⁵ ex GST

- Effective and rapid removal of blood and bodily fluids from surfaces
- Light fabric made from 100% polypropylene to facilitate bactericidal efficacy
- Strong and durable wipe
- Low linting
- Fragrance free



Why Choose Compostable Products?

Sustainable Solutions for Aged Care Facilities



Kidney Dish



4- Compartment Procedure Tray



Injection Tray

✔ Eco-Friendly:

Made from sugarcane bagasse, the EnvirOn Hollowware Range is 100% compostable, reducing landfill waste.

✔ Safe for Patients and Staff:

Free from harmful chemicals, ensuring a safer environment.

✔ Cost-Effective:

Decreases waste disposal costs and aligns with sustainability goals.

✔ High Quality:

Durable and reliable for everyday use in aged care facilities.

Switch to EnvirOn today and make a positive impact on the environment and your facility!



Scan to View Range
or visit www.sssaustralia.com.au

A brand by
DMUN

INVESTIGATING URINARY INCONTINENCE



When we get older, our body doesn't always work like it used to. It is estimated that 3 out of every 4 Australians who are in supported living situations have severe incontinence and 87% of people considering aged care list incontinence as a deciding factor. The risk increases with age, with 5% of people between the ages of 65 and 84 dealing with severe incontinence increasing to five times that when 85 or above.

Quality of life is greatly affected by incontinence. There is an increased risk of falls and fractures, which can lead to hospital beds and IAD scenarios. Social isolation and depression can result in lifestyle changes and when higher levels of care are needed, financial burdens increase.

Understanding the diverse subtypes of urinary incontinence is crucial as their causes and treatment approaches may vary. A systematic approach to addressing urinary incontinence involves thorough history-taking, physical examination, relevant tests, medication review, and excluding reversible causes.

There are several ways to investigate the effects of suspected urinary incontinence according to the RACGP. By using a three-day bladder chart, and recording input and output volumes, voiding times, fluid intake, and incidents of incontinence, Health practitioners get an approximate understanding of the incontinence severity. A GP can also request an MCS (microscopy, culture and sensitivity) urine collection test handled through pathology services or in complicated scenarios, health practitioners can request a Renal Ultrasound which can detect issues within soft tissue areas around the kidneys and bladder.

RACGP Clinical Guidelines State That Basic Investigations Into Urinary Incontinence Should Include A Portable Bladder Scan.

A portable bladder scanner is a quick and easy-to-use diagnostic device for the measurement of post-void residual urine (PVR). It can also detect how much fluid a bladder can hold, urinary retention, the shape of the bladder and the thickness of the bladder walls. Doctors can better determine the need for catheterization, and this results in fewer unnecessary placements of urinary catheters.

The RACGP mentions that the typical bladder capacity is approximately 500 mL, with little to no residual urine after voiding. However, a residual urine volume exceeding 100 mL may necessitate additional investigation.

Bladder scanners work with a transducer probe which generates ultrasound waves. These waves are bounced off the bladder and returned to the device for decoding. The screen on the scanner then shows a 3D interpretation of the patient's bladder.



Bladder Scanner + Touch Screen + Printer Kaixin BVT01

The portable bladder scanner BVT01 is a non-invasive ultrasound scanner used to measure bladder volume for assessing urinary retention and urinary incontinence. It aims to reduce the conduction time urinary frequency and reduce the risk of urinary tract infection and for treatment of urinary system diseases.

Non-invasive ultrasound scanner used to measure bladder volume for assessing urinary retention and urinary incontinence. Features smart and compact design with touchscreen, simple operation and advanced calculating method ensures the correct result for every anomaly shape.

- Advanced calculating method ensures the correct result for every anomaly shape
- Auto interpretation & easy manual note for amendment
- Simple Multi patient selection for different patient types
- Simple operation with less than 4 second scan time
- Smart & compact design with touchscreen
- Battery running time of > 9 hours
- Easy and fast information input
- Free online software upgrades
- USB port for data output
- Internal thermal printer
- Colour touch LCD

Item #: 1428101 Each

\$5,499 ex GST



Rolling Stand To Suit Kaixin BVT01 Bladder Scanner

Rolling Stand For Bladder Scanner BVT01 and BVT02. Assembly required.

Item #: 1428128 Each

\$994¹⁵ ex GST

HARTMANN



Prevention is the cure.



MoliCare® Skin

A daily skin care routine can help prevent painful skin conditions for those in your care, and save you valuable time every day.¹

- + Protects skin
- + Helps prevent irritation
- + pH skin balanced



Item #	Product Name	Price	ex GST	Category
1317482	Moist Skin Care Tissues	\$8 ⁸⁵	ex GST	P50
1377167	Barrier Cream	\$15 ⁰⁵	ex GST	Each
1317520	Body Lotion	\$12 ⁴⁵	ex GST	Each

1. Beekman D et al (2020) Best practice recommendations for holistic strategies to promote and maintain skin integrity. Wounds International.

NEW TO SSS AUSTRALIA



Optifast VLCD Shake 53g

The OPTIFAST VLCD Shakes range provide complete nutrition and come in a variety of great tasting flavours for you to enjoy.

Every shake contains 28 essential vitamins and minerals to assist with safe and effective weight loss on the OPTIFAST Program. High in protein and a source of fibre, these shakes offer convenient nutritious drinks that can be consumed in a place of a meal.

Item #	Flavour	Price	ex GST	Category
1523759	Mocha	\$50 ²⁵	ex GST	P12
1523694	Strawberry	\$50 ²⁵	ex GST	B12
1523686	Chocolate	\$50 ²⁵	ex GST	B12
1523678	Coffee	\$50 ²⁵	ex GST	P12
1523775	Coffee	\$71 ⁵⁰	ex GST	B18
1523767	Vanilla	\$71 ⁵⁰	ex GST	B18



QV Dermcare Daily Wash

QV Dermcare Eczema Daily Wash is a hydrating soap-free wash, clinically tested to gently cleanse and help soothe eczema skin. Scientifically formulated with a researched blend of ingredients including, Glycerin, Lactic Acid, Niacinamide (Vitamin B3) and a Ceramide blend.

Item #	Size	Price	ex GST	Category
1523473	350ml	\$20 ⁹⁰	ex GST	Each
1523481	1kg	\$34 ⁹⁰	ex GST	Each



QV Dermcare Daily Cream

The dermatologically tested moisturising cream is unlike most products recommended for eczema affected skin. Most are cosmetic products that are only designed for normal, healthy looking skin, QV Dermcare Eczema Daily Cream is clinically tested specifically for mild to moderate eczema skin.

Item #	Size	Price	ex GST	Category
1523600	350ml	\$23 ¹⁰	ex GST	Each
1523465	1kg	\$37 ⁴⁵	ex GST	Each



QV Dermcare Sting-free Ointment

QV Dermcare Sting-Free Ointment is a water-free formula that helps provide hydration whilst soothing dry, cracked skin without stinging, for up to 24 hours. Formulated with Ceramides EOP and NP, which are crucial components of the skin barrier, have been shown to support overall skin barrier function.

Item #	Size	Price	ex GST	Category
1523554	100g	\$11 ⁰⁵	ex GST	P6
1523562	200g	\$16 ⁹⁰	ex GST	Each



PROTECTION FOR SENSITIVE SKIN

Aqium Gel Moisturising 1L

- Quick and convenient way to sanitise your hands on the go
- Non sticky and air dries quickly so you don't need to wash, rinse or dry your hands
- Contains moisturisers to help leave your hands soft and refreshed

Item #: 1512064	Each
\$16⁷⁵	ex GST

Ego QV Intensive Moisture Cleanser 500ml

- Free from preservatives, fragrance, colour and lanolin to suit sensitive skin types
- Suitable for eczema, dry, sensitive and itchy skin
- Helps to maintain skin's protective properties and doesn't strip away natural oils
- Vegan Friendly - contains no animal derived ingredients

Item #: 1271643	Each
\$18⁷⁰	ex GST

Ego QV Intensive Cream 500ml Pump

- Triple-moisturising action, including 10% glycerin, for optimal skin hydration without a greasy residue
- Helps maintain the skin's natural protective barrier and lock in moisture
- Low-irritant formulation, fragrance, lanolin, propylene glycol and color free

Item #: 1420402	Each
\$20²⁰	ex GST

Cutimed® Sorbact®

Safe and effective infection management



Due to the **purely physical mode of action**, Cutimed® Sorbact® binds and removes bacteria without the release of any antimicrobial agents. So experience the effectiveness of **Cutimed® Sorbact®**. **Together we can make infection management safe in every respect.**

- For all contaminated or infected hard to heal wounds eg. Venous and diabetic ulcers
- Reduces the bacterial load and supports the natural wound healing process.
- No undesirable side effects, no cytotoxicity or development of bacterial resistance.

SHOP NOW 



Safe
Binds bacteria and fungi

Easy
No known contra-indications



DIABETES CARE



Freestyle Optium Exceed Blood Glucose Neo Monitor Kit

Introducing the FreeStyle Optium Neo System, with a choice of support tools especially designed for people on insulin. Finally a meter that's as easy or as useful as you want it to be.

Item #: 1196440 Each
\$67⁶⁰ GST FREE



GlucoKey Blood Glucose Monitor Without Test Strips

The all new GlucoKey offers a compact and efficient means of testing blood glucose and B-Ketone all from the same device. The meter allows for ease of use through a back-light for night-time use and contoured strip insertion point.

Item #: 1417207 Each
\$34⁸⁰ GST FREE



Freestyle Optium

- No coding required
- Small blood sample required
- Short second test time
- Top or end fill with visual confirmation
- Foil wrapped strips help protect from air and moisture
- Fill trigger ensures test starts only when sufficient blood sample has been applied

Item #	Description	Price	Tax	Code
1196431	Glucose Electrode Strips	\$56⁵⁵	GST FREE	B100
1307886	B-Ketone Test Strips	\$6⁸⁵	GST FREE	B10



GlucoKey Level 1 Control Solution
GlucoKey Level 1 Control Solution for Glucose Test Strips is the low range control solution to check your GlucoKey meter and strips with. The range is displayed on the side of your GlucoKey test strip vial.

Item #: 1490788 Each
\$5⁶⁰ GST FREE



GlucoKey Test Strips
To be used with Glucokey and Glucokey Connect blood glucose machines

Item #: 1416065 B100
\$40²⁵ ex GST



Accu-Chek Safe-T-Pro Plus Lancet B200

Multiple safety features combined with customised settings make the Accu-Chek Safe-T-Pro Plus lancing device especially advantageous. This single use lancing device supports patient safety and ease of use. It also complies with the safety requirements demanded by health authorities, legislators, and health institutions.

Item #: 1161392 B200
\$49⁵⁶ GST FREE



Gloop Medication Lubricant Raspberry Low Sugar 500ml

Gloop® is the first and only purpose designed medication lubricant available in Australia that negates the need to crush or alter medications, has no known drug interactions, and is designed to break down immediately once in the stomach, therefore having minimal or no impact on absorption

Item #: 1406876 Each
\$37²⁰ ex GST



NUTREN Diabetes Vanilla Ready to Drink 200ml

Nutren Diabetes Powder contains essential nutrients, including 30 vitamins and minerals, to meet the nutritional needs of people living with diabetes. It can be used for both oral and tube feeding.

Item #: 1448846 C24
\$92⁷⁵ GST FREE



Glucojel 1 kg bag Jelly Beans

Australia's best tasting jelly beans! Glucojel jelly beans have been a national institution for over 60 years.

Item #: 1399160 Each
\$15¹⁵ ex GST

FACILITY SET UP

FACILITY SET UP



ALL YOUR FACILITY SET UP NEEDS
FROM ONE GREAT SUPPLIER

VITAL SIGNS
MONITORS

VISITOR
CHAIRS

SYRINGE
DRIVERS

WHEELCHAIRS

BEDS

LINEN &
PILLOWS

PILL
CRUSHERS

PRESSURE
MATTRESSES

TILT
BEDS

BEDROOM
FURNITURE

FIND OUT MORE!



NURSE CALL
ALARMS

SYRINGE PUMP



BD BodyGuard T Syringe Pump

The BD BodyGuard T Syringe Pump is a medical device used to administer precise and controlled amounts of fluids, medications, or nutrients to patients intravenously, subcutaneously, or epidurally.

The BD BodyGuard T Syringe Driver consists of a syringe holder that accommodates various sizes of syringes and a mechanism that slowly pushes the plunger of the syringe to deliver the desired amount of liquid at a controlled rate.

The BD BodyGuard syringe pump is programmable, allowing healthcare providers to set specific infusion rates and dosage parameters, ensuring accurate and steady delivery of medications or fluids over a specified period.

Item #: 1453157 Each
\$3,016 GST FREE



9V Battery Model 6LR61 for BodyGuard T

Item #: 1451316 B10
\$42²⁵ ex GST



Lockbox for Niki T34 & BodyGuard T Syringe Pump

Item #: 1264077 Each
\$182 GST FREE



Lockbox Carry Bag for Niki & BodyGuard T Syringe Pump

Item #: 1264068 Each
\$128⁴⁵ GST FREE



Label Line Subcutaneous Brown

Item #: 1309021 Roll
\$23²⁵ ex GST



Label Subcutaneous Use 60mm x 50mm Brown

Item #: 1330381 Roll
\$28⁹⁵ ex GST

SPEECH PATHOLOGY



Amplivox 116 Stand Alone Screening Audiometer

Designed with portability and reliability in mind, the Model 116 is a portable, manual screening audiometer designed for quick and efficient audiometric testing.

The lightweight and robust design benefits any audiologist who travels to conduct pure tone, warble and continuous hearing tests. The soft key control panel is dependable and functions with ease, with a clear display delivering precise information.

Strong and reliable, offering the simplicity needed to conduct hearing tests in the most challenging environments. At only 720g (1.58lbs), it is one of the lightest portable audiometers available.

Item #: 1437453 Each
\$2,802⁸⁵ ex GST



Electronica 800M PC Base Audiometer

The Electronica 800M is a PC Based Screening Audiometer that performs Air Conduction tests. It is connected by USB to a PC and operate, simple to install, and intuitive software. It includes the Radioear DD45 headphones & Peltor Audio Cups and has a robust patient response switch ensuring a smooth user experience. It is a suitable device for General Practices, Occupational Clinics, Audiologists, Pre Employment & Schools plus many more.

Item #: 1446150 Each
\$2,999 ex GST



Thickenup Gel Express Unflavoured 450ml

THICKENUP® Gel Express can help to improve the safety of swallow by increasing viscosity, in line with IDDSI¹ guidelines (levels 1 to 4). Its dosing system allows for quick and accurate dispensing, delivering multiple servings faster.

Item #: 1456970 C8
\$236⁵⁰ GST FREE

Not only is THICKENUP® Gel Express easy to use, it also reaches the desired viscosity faster than powder-based thickeners[†] in complex liquids such as milk, orange juice and oral nutrition supplements.



Durable protection for at-risk or severely dry skin

3M™ Cavilon™ Durable Barrier Cream

Formulated with 3M polymer technology, 3M™ Cavilon™ Durable Barrier Cream is a concentrated, fragrance-free moisturising skin barrier that resists wash-off and allows adhesives to adhere.¹ It's ideal for moisturisation of at-risk and/or severely dry skin. It also provides a barrier which protects skin at low risk for breaking down.¹

Item Code	Size	Price	Qty
1299174	28g Tube	\$9 ³⁵ GST FREE	Each
1298720	92g Tube	\$26 ⁰⁵ GST FREE	Each



Reference: 1.3M Data on file. 3M Australia Pty Limited. ABN 90 000100 096. 3M.com.au/Cavilon. © Solventum 2024. Solventum and the S logo are trademarks of Solventum or its affiliates. Other trademarks are the property of their respective owners.



Precise Thick-N INSTANT

Precise Thick-N INSTANT is the GOLD standard for thickening beverages for people with dysphagia. This innovative liquid thickener has been welcomed by the industry as it meets the need for a convenient, accurate way to thicken beverages to precise and predetermined viscosities.

Item #	Size	Price	GST	Code
1337467	Single Serve 500ml	\$293 ⁴⁰	GST FREE	C10
1321382	Single Serve 3L	\$204 ⁶⁰	GST FREE	C2
1321390	Bulk Serve 3L	\$204 ⁶⁰	GST FREE	C2



Resource Thickenup Clear

THICKENUP® Clear is the clear solution for dysphagia. Our exclusive patented xanthan gum formulation offers a clear solution for patients and clinicians to thicken a wide range of hot and cold drinks and foods.

Item #	Size	Price	GST	Code
1290606	1.2g Sachets	\$11 ⁰⁵	GST FREE	P24
1281682	125g	\$12	GST FREE	Each
1281690	900g	\$71 ⁶⁵	GST FREE	Each



W.A Cardio Suite ECG AM12 Wired Acquisition Module

The Welch Allyn Diagnostic Cardiology Suite now includes ECG and spirometry capabilities, helping you shorten the path to diagnosis for improved patient outcomes. Simplify clinical workflows, reduce manual data entry and minimize complexity with one device that can perform two reimbursable tests. Our flexible platform allows you to adapt the Diagnostic Cardiology Suite to meet your clinical needs—whether you want to perform both exam types, or prefer a standalone ECG or spirometry solution. We're combining operational experience with clinical excellence to bring you one solution for two critical exams—because simple is smart.

Item #: 1438700 Each

\$2,499 GST FREE



W.A CP150 Interpretive 12 Lead Resting ECG Touch Screen

The Welch Allyn CP 150 Resting Electrocardiograph (ECG), paired with an optional spirometry feature, merges classic box ECG functionalities—such as comprehensive printouts and clinical decision support—with user-friendly innovations. The CP 150 Resting Electrocardiograph's intuitive touchscreen interface, swift one-button operation, and adaptable connectivity choices aim to bolster staff efficiency and ensure precise test results. The addition of the optional spirometry module not only streamlines processes but also enables two reimbursable tests using a single device. This group of features produces a device designed to optimize time management, streamline workflows, and facilitate efficient patient information management within your facility.

Item #: 1314270 Each

\$4,199 GST FREE



Medico ECG Electrodes

Superior silver/silver chloride technology and quality construction produce an electrode that performs every time. Excellent adhesion and clear traces mean no repeat testing.

Item #	Description	Price	Tax	Code
1524038	Resting Tab Electrode	\$11⁷⁵	ex GST	P100
1524011	Stress Monitoring Electrode	\$15⁰⁰	ex GST	P60
1524003	Resting 44x28mm Banana Electrode	\$27⁷⁵	ex GST	P100

ZINC OXIDE & SKIN INTEGRITY



Historically, topical zinc has been used in the treatment of wounds for over 3,000 years. Zinc is a metallic trace element that is essential for good health and is second only to iron as the most abundant element in a healthy human's body. Roughly one-fifth of our total zinc content is found within our skin (Maher, 2015) and it plays an important role in the body in our immune system, antibody production, cell structure and integrity, tissue maintenance, and wound healing (Kogan, et al, 2017). So, it makes sense that if we are deficient in zinc, wound healing could be compromised.

The Benefits of Zinc Oxide - Healing with Paste Bandages

Zinc bandages are used primarily in lower limb wound management to treat leg ulcerations and the skin conditions associated with a Venous leg ulcer. They are commonly applied to help manage Varicose Eczema and Hyperkeratosis. Zinc bandages can also be used in the management of skin conditions related to chronic oedema and for the treatment of patients with a diagnosis of Red Leg Syndrome.

VISCOPASTE PB7 bandages deliver a protective skin barrier, reduce inflammation and provide a moist healing environment when the skin's integrity has been compromised (Williams, 1999; Lansdown et al, 2007). The paste bandages are soothing and cooling when in situ as the zinc oxide absorbs into the wound or skin surface, which may help to correct a local zinc deficit and aid healing. (Agren 1990). Patch testing for 48 hours is recommended before initial use.

VISCOPASTE PB7 bandages can be used as an adjunct to a compression therapy system* or as the primary wound contact layer. Stacey et al (1997) found in trials that the application of a paste bandage when used in combination with compression therapy significantly improved venous leg healing rates. Each bandage is single-use and can be left in place for up to 7 days of wear and provides a cost-effective treatment.

*A holistic assessment is required before the selection and application of an appropriate compression therapy.

VISCOPASTE PB7 bandages are made of cotton and do not contain elastic fibres, meaning they must be applied in a way that allows for swelling which may occur during the inflammatory phase of healing.

Clinicians should use one of the following methods of application:

- pleating the bandage following each turn
- cutting the bandage after each turn
- cutting the bandage into strips before applying

On removal of the bandage, always unwind and discard, do not cut. To be used under the supervision of a Clinician.



Viscopaste Impregnated Bandages 7.5cm x 6m

VISCOPASTE PB7 BANDAGE 7.5cm x 6m is made from open woven cloth which is impregnated with a paste containing 10% Zinc Oxide. The bandage contains no elastic fibres and does not stretch. **VISCOPASTE** may be used as a primary dressing over the wound site or skin condition and may be covered with an occlusive bandage or garment.

Item #: 1002180 Each
\$21¹⁰ ex GST



Zipzoc Zinc Oxide Impregnated Bandages Stocking Sachets

ZIPZOC is a sterile, single use, 20% zinc oxide impregnated stocking which is used widely in wound management and dermatology to provide patients with an aid to the treatment of chronic leg ulcers, eczema, dermatitis, cellulitis and hyperkeratosis.

Item #: 1168366 B4
\$106²⁰ ex GST



Bena Zinc Oxide 10% Spray 100ml

Abena A/S Zinc Oxide Cream Spray is available in 100ml capacity which protects and regenerates the skin. This spray promotes a natural skin restoration for up to 24 hours.

Item #: 1456857 Each
\$25⁹⁵ ex GST



The latest in care.

Our new range of **Detmold Medical** Tri-Panel respirator masks are the perfect fit for health care providers and in clinical environments. We also offer a range of single-use respirators and masks, to help you take care of those who care. **Designed, tested and manufactured in Australia.**



Level 3 N95 Surgical Tri-Panel Respirator

Item #: 1521217 B20
\$39⁹⁵ ex GST



P2 N95 Medical Tri-Panel Respirator

Item #: 1521209 B25
\$49⁹⁵ ex GST





Omron HEM7155T Plus Dual User Blood Pressure Monitor

- Omron Connect App makes it easy to save, track and manage your health data
- Average for last 3 readings within 10 minutes
- 2x users x 60 measurements in memory
- Irregular heartbeat detection
- Body movement detection
- Intellisense technology
- Hypertension indicator
- Bluetooth connectivity
- Low battery indication
- Cuff wrapping guide
- Guest mode

Item #: 1474766 Each
\$194 ex GST



Thermometer Clinical Non Contact Infrared A200

- Memory function allows you to recall previous results up to 25 previous results
- Easy to read LCD with green back light in a dark environment
- Light weight and suitable for all ages

Item #: 1430475 Each
\$69 ex GST



Welcare PureAir Desktop Air Purifier

- Suitable for desktops and small rooms 12.36m²
- Dust sensor & air quality indicator (4 colours)
- Adjustable fan speed - 4 levels (one button)
- Timer function 1hr-2hr-4hr-8hr-continuous
- Filters dust, pollen, smoke and bacteria
- Aromatherapy function
- Low-noise operation
- Removes odours

Item #: 1489852 Each
\$159 ex GST



HeartSure Fingertip Pulse Oximeter

- Measures Oxygen Saturation (SpO₂)
- Measures Pulse Rate (PR)
- Dual colour OLED display
- Multi-directional display
- Low perfusion: 0.3%
- 6 display modes
- Alarm function
- 2 year warranty

Item #: 1339931 Each
\$75 ex GST



Welch Allyn Connex Spot Vital Signs Monitor NIBP/Nonin SpO₂/Pro6000

- Automated Early Warning Scoring to assist clinicians identify and respond to signs of patient deterioration (ie. MEWS, NEWS)
- Secure FIPS-approved encryption of transmitted data and a range of additional security features
- Personalised work processes for recording patient observations and vitals options (ie. BP cuff location)
- Multiple mounting options with mobile stands and wall mounts to suit specific work environments
- Single sign-on options - including integration with Imprivata - available for quick, secure clinician login
- SureTemp® Plus Oral / Axillary or Braun ThermoScan® PRO 6000 Tympanic Thermometry
- Options to use the leading SpO₂ systems from Masimo®, Nellcor® or Nonin®
- Options for WiFi®, Ethernet and Bluetooth® connectivity to EMR systems
- Correct hypertension diagnosis using blood pressure averaging
- Blood pressure readings in 15 seconds using the SureBP®
- Connectivity to Detecto, Health o meter® and SECA scales
- Two-year warranty

Item #: 1347101 Each
\$2,400 ex GST



Basket Stand
 Item #: 1348361 Each
\$569 ex GST



Work Stand
 Item #: 1396501 Each
\$850 ex GST

Surfex®

Dry Surface Biofilm Remover and Disinfectant

Removes the **Biofilm,**
 +
 Kills the **Pathogens.**

Your **2-in-1** solution!

The best choice for staff safety



Whiteley
 Protecting lives and livelihoods since 1933

SHOP NOW



Healthcare Associated Infections (HAIs) impact over 200,000* Australians every year. One reason is that pathogens lie dormant under biofilm, which other disinfectants don't remove.

Surfex® is the world's first proven dry surface biofilm remover and disinfectant. Surfex® effectively removes biofilm and kills pathogens such as COVID-19, MRSA, VRE/CRE, Candida albicans, Candida auris and Norovirus.

Proudly Australian made, you can rely on Surfex® to protect your staff, your residents and your reputation.

Greg Whiteley
 Dr. Greg S. Whiteley

ARTG 257360

To trial the World's First proven Dry Surface Biofilm Remover, call 1800 833 566 or go to whiteley.com.au

* Cruickshank M & Ferguson J [eds] (2008) Reducing Harm to Patients from Health Care Associated Infection: The Role of Surveillance. Australian Commission for Safety and Quality in Health Care, p3.



Welch Allyn DS66 Sphygmomanometer with Flexiport 4-Cuff Kit

This innovative gauge features a smooth trigger-style air release valve and ergonomic design with soft plastic inserts at key touch points for comfort and easy operation. With gear-free DuraShock™ technology, you can count on the robust design of the DS66 Trigger Aneroid to remain in calibration longer than traditional aneroids, providing more accurate readings over time and a lower lifetime cost, helping to improve practice efficiencies and your bottom line.

- Gear-free DuraShock™ technology
- Certified accuracy to +/- 3 mmHg
- Unsurpassed reliability with a unique trigger-release design
- Withstands a 30" drop
- Laser-engraved dial face for accuracy
- Jeweled movement for long life
- Soft comfort grip and trigger deflation valve for smooth, one-finger operation
- 15-year calibration warranty
- Latex-free for safety

Item #: 1222101 Each
\$289 ex GST



Welch Allyn ThermoScan PRO6000 w/ Small Cradle

The Welch Allyn Braun ThermoScan PRO 6000 ear thermometer featuring advanced measurement technology, has the speed and simplicity you need to capture accurate and reliable temperature readings in a comfortable way for patients. The Braun PRO 6000 advantage lies in our innovative PerfectTemp technology, which overcomes the potential for low readings compared to core, by adjusting for factors that impact measurement accuracy. The sleek design is easy to disinfect and upgraded security features help protect and maximize your investment. Memory recall button and 60-second pulse timer are added features to support staff efficiency and help improve workflow.

- Innovative PerfectTemp™ technology adjusts for variability in probe placement
- ExacTemp™ technology detects stability of the probe during measurement
- Designed to take quick and accurate readings, to improve patient satisfaction
- 60 second pulse timer assists with manual measurement of pulse rate, respiration
- Electronic security features help prevent theft, loss
- Small cradle for probe cover storage that pop-up for easy attachment
- 3 Year Warranty

Item #: 1337564 Each
\$249 ex GST



Shower Commode Aspire Tilt & Recline

- 30° of backrest angle adjustment in 10° increments for effortless repositioning and maximum user comfort
- -5° to 35° of gas-assisted tilt-in-space functionality minimises manual handling efforts during the showering process
- 4x total locking stainless steel castors provide brake access from all angles
- Open back straps provide greater access for bathing needs
- Backrest may be configured as full length or 3/4 length to accommodate an advanced headrest
- Open armrests style accommodates a wider sitting surface whilst maintaining a smaller base footprint for improved accessibility
- Height adjustable, swing away footrests with calf pads for optimal positioning, support and ease of transfers
- Tool-free height adjustment to 4 settings allow the commode to be easily adapted to varying needs

Specifications

Overall Height	1315 - 1485mm
Overall Width	560mm
Backrest Height	900mm
Armrest Height	275mm
Anterior Seat Height	460 - 630mm
Posterior Seat Height	460 - 620mm
Safe Working Load	250kg
Seat Width	460mm

Item #: 1368540 Each
\$3,114⁵⁵ GST FREE



Clearance PPE & Dressings Products


Item #	Details	Price	Tax	Unit
1437666	Medical Goggle DSU	\$4	ex GST	Each
1457268	Disposable Face Shield	\$0 ⁸⁹	ex GST	Each
1272041	Leukoplast Standard 2.5cm x 5m	\$4 ²⁰	ex GST	Each
1416618	Dressing Wound Large Cotton Pad w/ Cotton Bandage	\$7 ⁵⁰	ex GST	P6
1425862	Hydrocoll Dressings Sterile 5 x 5cm	\$55	GST FREE	B10
1480464	Surgipore Non Woven Tape 5 x 9.14cm	\$9	ex GST	B6
1415107	Surgical Face Mask w/ Ties Anti Fog Level 3	\$7 ⁵⁰	ex GST	B50

A perfect fit.
 Every time.

Biatain® Silicone with 3DFit Technology reduces exudate pooling and thereby the risk of infection

If a gap forms between a dressing and the wound bed it can impact the healing. It is here that excess exudate can pool, leading to bacterial growth, leakage and potentially delayed healing.

However, there is a simple solution to this common challenge. Thanks to 3DFit Technology, Biatain® Silicone conforms to the wound bed to fill the gap and reduce exudate pooling for optimal wound healing conditions.

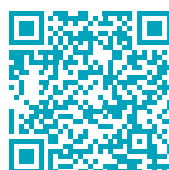

Scan the QR code to order stock

Coloplast Pty Ltd, PO Box 240, Mount Waverley, VIC 3149 Australia
 www.coloplast.com.au Coloplast and the Coloplast logo are trademarks of Coloplast A/S.
 ©2024-06 WOU548 Coloplast A/S. All rights reserved.

Biatain® Silicone



HiCare Superior Bathing System

The most efficient and hygienic bathing solution.

Developed in Australia our HiCare products are lab safety and dermatologically tested and suitable for people unable to do their own bathing.

Specialists in waterless bathing solutions.

Hospitals + Aged Care + Homecare + Disaster Relief

HiCare Waterless Shampoo

HiCare ecoBath

HiCare Bath

HiCare ECOBath

Economical Disposable Bathing Solution



SHOP NOW



SSS AUSTRALIA PRODUCT GUIDE

OUT NOW!

Talk to your personal
account manager or
add to your next order!

